



## DreamStation Modem Removal and Installation Instructions

**⚠ Please note: You must complete these steps for modem installation to ensure prescription delivery to your replacement device.**

These instructions will allow you to utilize your existing modem for your replacement device, or in the case you receive a replacement modem separately.


### You may have one of two different modem models.

Determine your current modem type and choose the instruction option that matches your modem.

*You will only need to use one instruction path based on your modem type.*


#### Option A: Standalone Modem Cartridge

\* Instructions below

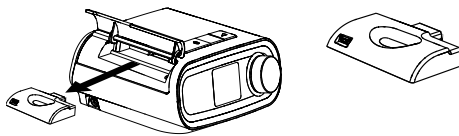


#### OR Option B: Combo Modem/Door Cartridge

\* Instructions on back page

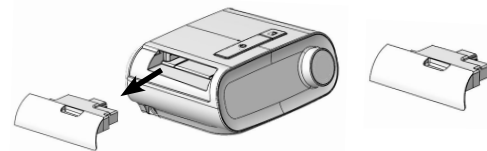

#### A: Instructions for Standalone Modem Cartridge Removal:


- 1 • Ensure the therapy device is off and not blowing air. You can keep the unit plugged in for this procedure.
- 2 • Open the modem door located on the left side of your current therapy device (FIG. A).  
 • Gently, pull the old modem out of your therapy device (FIG. B).  
 • Set modem aside for later instruction.  
 • See separate **"BASIC SETUP & RETURN"** instructions for proper return of your old device.

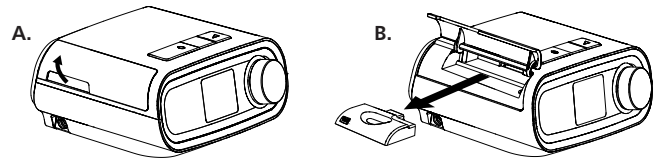

#### A: Instructions for Standalone Modem Cartridge Installation:

- 1 • Take the modem that you removed from your older DreamStation device.  
 • Open modem door (FIG C) on your replacement DreamStation device and slide the modem into place (FIG. D).  
 • Close modem door.  
 • Please keep the unit plugged in for at least 5 minutes to ensure your new modem connects successfully. This will happen automatically, and no additional steps are required.

**⚠ Please Note: You must complete these steps for modem installation to ensure prescription delivery to your replacement device.**

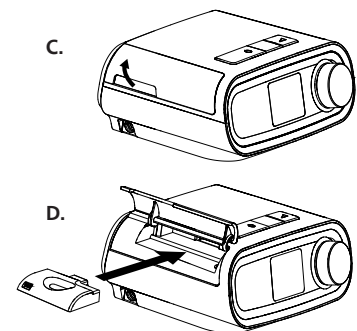


# DreamStation

## Modem Removal and Installation Instructions


### Option B: Combo Modem/Door Cartridge

\* Instructions below

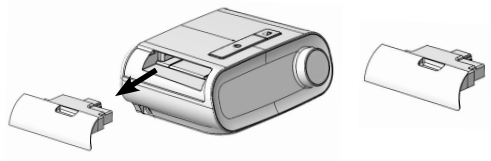



### B: Instructions for Combo Modem/Door Cartridge Removal:

- 1 • Ensure the therapy device is off, and not blowing air. You can keep the unit plugged in for the procedure.
- 2 • Gently, pull the old modem/door combo out of your current therapy device (**FIG. A**).  
• Set modem aside for later instruction.  
• See separate **“BASIC SETUP & RETURN”** instructions for proper return of your old device.

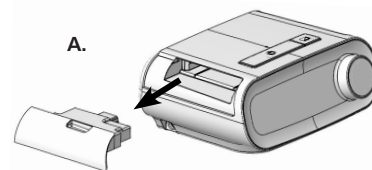

### B: Instructions for Combo Modem/Door Cartridge Installation:

- 1 • Your replacement DreamStation device will come with a modem door attached. You will have to remove it if you have a combo modem/door cartridge model.  
• Take the modem that you removed from your older DreamStation device or the replacement modem that may have been sent to you separately and use in the steps below.
- 2 • Remove the modem door from the left side of your replacement DreamStation device by lifting the door open and firmly sliding the door to the left (**step 1**) and pulling the right side of the door toward you until the hinge pops out (**step 2**). This may take a couple of attempts (**FIG. A**). Keep the door for possible future use.

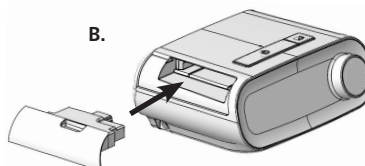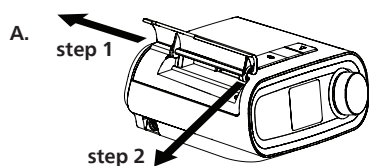

- Slide your existing or new modem into the accessory slot until it clicks into place (**FIG. B**).
- Please keep the unit plugged in for at least 5 minutes to ensure the modem connects successfully. This will happen automatically, and no additional steps are required.

① **Please Note: You must complete these steps for modem installation to ensure prescription delivery to your replacement device.**

Visit us at [www.philips.com/ds1-replacement](http://www.philips.com/ds1-replacement) for a digital copy of all of the materials, information about setting up this device, and an introduction to product features.

**Thank you for your cooperation in this effort.**

Please contact us at 833-262-1871 or [www.philips.com/SRC-update](http://www.philips.com/SRC-update) if you have any questions.